



New cartridge system  
**NEW ADHESIVE**



SOOO ADHERENT  
IS **MUCOPREN**<sup>®</sup> SOFT

**SOFT RELINING**




**KETTENBACHDENTAL**

Simply intelligent


# MUCOPREN® SOFT

## Prosthetic relining

Mucopren® Soft is a soft long-term relining material based on a silicone for **direct and indirect application**.



**Special long-term adhesion:**  
Excellent adhesion to acrylic dentures through patented Mucopren® Adhesive



**Unique sealing through its very smooth, hydrophobic silicone surface:**  
Inhibits bacterial colonization thus ensuring hygienic conditions on the prosthesis<sup>1)</sup>



Application video

### Simple treatment & fast processing

Faster setting times and shorter waiting times ensure a fast and smooth process in the practice and laboratory.

### Supports the healing process of implants

Also suitable for use in implantology for trouble-free osseointegration.



### High wearing comfort

The definitive resilient reline is completely odorless and tasteless and makes wearing the prosthesis as pleasant as possible.

### Cost-effective mixing tip

The new blue mixing tip has up to 20% less residual amount than the previous green mixing tip (Ø 6.5 mm) for Mucopren® Soft.



**Market leader** in the field of permanent soft reline materials<sup>3)</sup>



# HOLDS TO ITS PROMISES



## FURTHER INDICATIONS



- For the creation of **permanently soft relinings**
- For relieving pressure on the surgical site after insertion of **implants**
- For balancing mucous membrane areas with different resilience
- For relieving sharp bony ridges
- For relining **interim prostheses**
- As **mucous membrane conditioner** in case of prosthetic stomatitis
- For creating and relining **obturators**

## Benefits of using the new adhesive

» ONLY A **SINGLE APPLICATION** NECESSARY



### CHAIRSIDE

3 MIN

The processing time is reduced to only **3 minutes** following mouth removal. No water bath necessary.



### LABSIDE

3 MIN

The processing time is reduced to only **3 minutes** in the water bath or pressure pot.

<sup>1)</sup> Result from in-vitro studies in which it was shown that the growth of *Candida albicans* was prevented thanks to the smooth silicone surface





<sup>2)</sup> Compared to the previous green mixing tip for Mucopren® Soft (Ø 6.5 mm)

<sup>3)</sup> acc. to GfK (a German consumer research association)

# MUCOPREN® SOFT – THE RANGE



## Mucopren® Soft

<b>Mucopren® Soft Basic Set</b>	REF 28105	50 mL Mucopren® Soft, 50 mL silicone sealant, 10 mL adhesive, 6 blue mixing tips, 16 red mixing tips, 1 brush holder, 20 single-use brushes, 1 steel bur, accessories	 
<b>Mucopren® Soft Normal pack</b>	REF 15687	2 x 50 mL Mucopren® Soft, 6 blue mixing tips	
<b>Mucopren® Silicone sealant</b>	REF 15686	50 mL Mucopren® Silicone sealant, 8 red mixing tips	
<b>Mucopren® Adhesive</b>	REF 14203	10 mL Mucopren® Adhesive	



### Technical data

Product name	Mixing ratio	Working time at 73 °F ≤	Intraoral setting time at 95 °F ≥	Water bath	End of setting ≥	Shore hardness (approx.)
Mucopren® Soft – direct relining	1:1	2:15 min.	3:15 min.	Not applicable	5:30 min.*	A 28
Mucopren® Soft – indirect relining	1:1	2:15 min.	Not applicable	3:00 min.	> 3 min.	A 28
Mucopren® Soft silicone sealant	1:1	1:30 min.	Not applicable	Not applicable	6:30 min. at 23°C	A 33

\*Total setting time (removal time from mouth) from start of mixing

## NOTES

---



---



---



---



---

Kettenbach SNC  
7, Place de la Gare  
57200 Sarreguemines · France  
info@kettenbach.com  
www.kettenbach-dental.com

Not for use, sale and distribution in Germany.



**KETTENBACHDENTAL**  
Simply intelligent

1410871\_1024

